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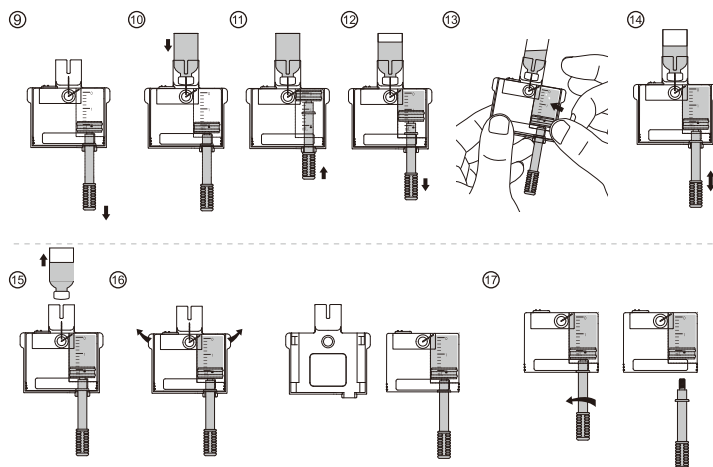
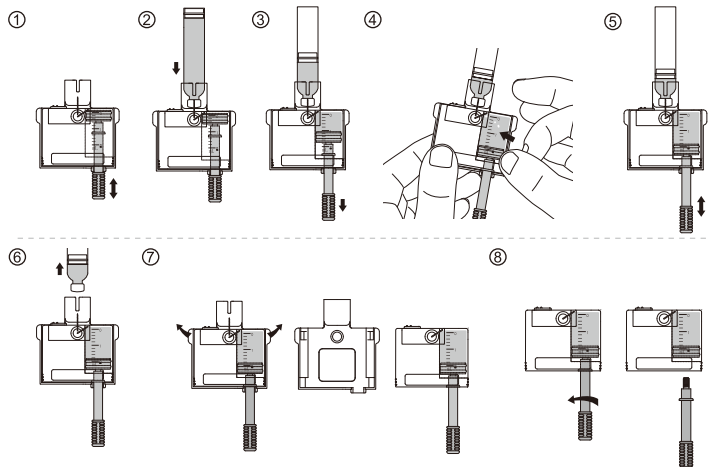
REF MTM-3

Microtech Medical (Hangzhou) Co., Ltd.
No. 108 Liuzhe St., Cangqian, Yuhang District,
Hangzhou, 311121 Zhejiang, Chiny
www.microtechmd.com

CMC Medical Devices & Drugs S.L
C/Horacio Lengo N° 18 CP 29006,
Málaga-Spain

Importer:
Synoptis Industrial Sp. z o.o.
ul. Forteczna 35-37
87-100 Toruń

1001-PMTL-272. V04

PL Polski

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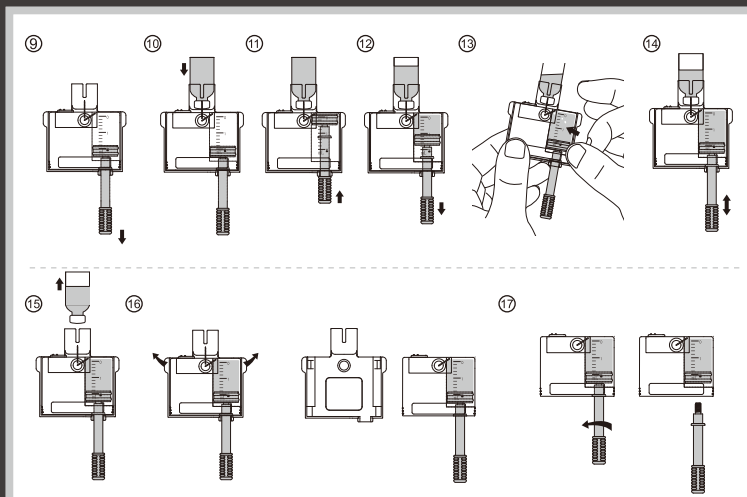
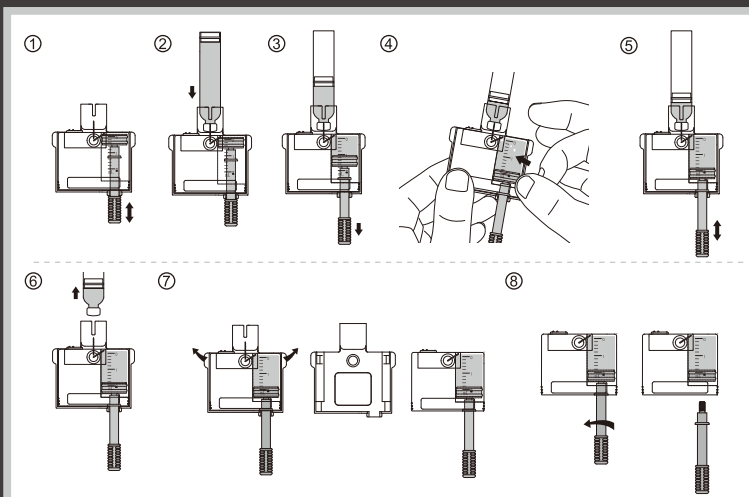
1001-PMTL-272. V04

English

Product Name: Ambulatory Insulin Infusion Pump Reservoir

REF: MTM-3

Shelf Life: 3 years

Manufacturer Date: Check label on box

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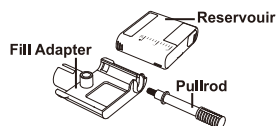
2. Components

The Reservoir package consists of a reservoir, fill adapter and pullrod, as shown in the figure below:

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4. Contraindication

- 1 Use only with U-100 insulin.
- 2 Cannot be used for storage or infusion of blood or blood products.

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6. Usage

The fill adapter for 3 mL insulin cartridge:

- 1 Remove the new reservoir from the packaging.
- 2 Use the pullrod to pull the piston to the full position inside the reservoir, then push it

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- 3 Use an alcohol wipe to clean the top of the insulin vial and then attach it to the fill adapter (Figure 2).

- 4 With the insulin vial on top, insert the needle into the vial, slowly pull the pullrod and draw insulin into the reservoir (Figure 3).

- 5 Tap the side of the reservoir to encourage any air bubbles to rise to the top of the reservoir (Figure 4).

- 6 Push the pullrod slowly to eject the bubbles back into the vial, then slowly pull to draw more insulin into the reservoir. Repeat until the reservoir is filled with the required amount of insulin again (Figure 5).

- 7 Remove the insulin vial from the fill adapter (Figure 6).

- 8 Pull the two release tabs away from the reservoir to release the fill adapter (Figure 7).

- 9 Remove the pullrod by unscrewing it counterclockwise (Figure 8).

The fill adapter for 10 mL insulin vial:

- 1 Remove the new reservoir from the packaging.

- 2 Use the pullrod to pull the piston to the full position inside the reservoir (Figure 9).

- 3 Use an alcohol wipe to clean the top of the insulin vial and then attach it to the fill adapter (Figure 10).

- 4 Insert the needle into the vial and inject the air, injecting air makes it easier to withdraw insulin from the vial (Figure 11).

- 5 Slowly pull the pullrod downwards in the direction of the arrow to fill the reservoir with insulin (Figure 12).

- 6 Tap the side of the reservoir to encourage any air bubbles to rise to the top of the reservoir (Figure 13).

- 7 Push the pullrod slowly to eject the bubbles back into the vial, then slowly pull to draw more insulin into the reservoir. Repeat until the reservoir is filled with the required amount of insulin again and there are no any air bubbles (Figure 14).

- 8 Remove the insulin vial from the fill adapter (Figure 15).

- 9 Release the reservoir from the fill adapter (Figure 16).

- 10 Remove the pullrod by unscrewing it counterclockwise (Figure 17).

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Excess air in the Reservoir may cause inaccurate delivery of insulin. Eject as much air as possible.

It is suggested that the product should be changed every three days.

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9. Storage Condition

This product should be stored in a non corrosive gas, cool, dry, well ventilated area. Avoid storing with the chlorinated disinfectant.

10. Symbol list

Single Use Only	
Caution	
Expiration Date	
Lot Number	
Sterilized by Ethylene Oxide	
Read Instructions	
Do Not Use if Package is Damaged	
Manufacture Date	
Manufacturer	

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EN For use with the Equil Patch Insulin Pump System

Microtech Medical (Hangzhou) Co., Ltd.
No. 108 Liuze St., Cangqian, Yuhang District,
Hangzhou, 311121 Zhejiang, China
www.microtechmd.com

CMC Medical Devices & Drugs S.L
C/Horacio Lengo Nº 18 CP 29006,
Málaga-Spain

Importer:
Synoptis Industrial Sp. z o.o.
ul. Forteczna 35-37
87-100 Toruń

1001-PMTL-272_V04

MicroTech
Medical


CE 0197

equil™

Insulin Reservoir

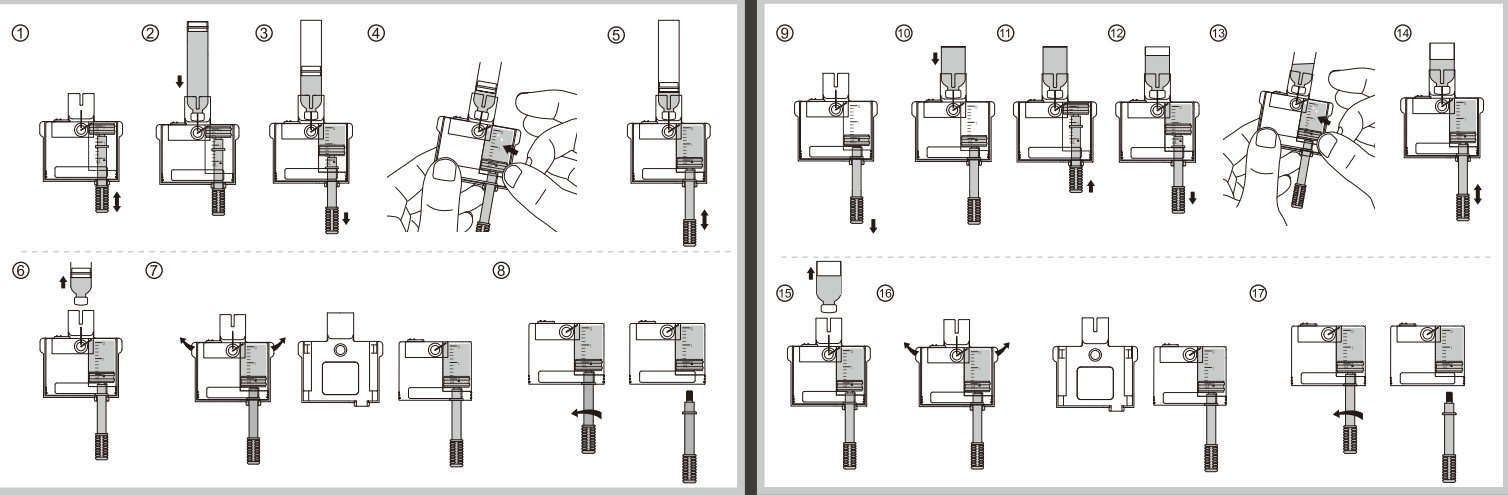REF MTM-3

EN English

Product Name: Ambulatory Insulin Infusion Pump Reservoir
REF: MTM-3
Shelf Life: 3 years
Manufacturer Date: Check label on box

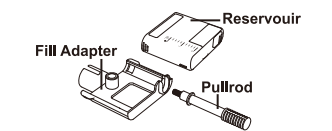
1. Purpose

The Equil Reservoir is a component of the Equil patch insulin pump system and is used to store insulin.

The sterilized reservoir should be used with the Equil patch pump and disposable infusion set produced by MicroTech Medical.

2. Components

The Reservoir package consists of a reservoir, fill adapter and plunger, as shown in the figure below:

The total capacity of the reservoir is 2.0ml, which is equal to 200 units of U-100 insulin.

3. Indications

This product is indicated for the subcutaneous infusion of insulin using the Equil Patch Insulin Pump (MTM-1).

4. Contraindication

- Use only with U-100 insulin.
- Cannot be used for storage or infusion of blood or blood products.

5. Application

This disposable product is compatible with the Equil Patch Insulin Pump System and is suitable for treating diabetes requiring short-term or long-term insulin pump therapy.

6. Usage

- The fill adapter for 3 ml insulin cartridge:
 - Remove the new reservoir from the packaging.
 - Use the plunger to pull the piston to the full position inside the reservoir, then push it

completely to the empty position. Repeat this three times ending with the piston in the empty position (Figure 1).

- Use an alcohol wipe to clean the top of the insulin vial and then attach it to the fill adapter (Figure 2).

- With the insulin vial on top, insert the needle into the vial, slowly pull the plunger and draw insulin into the reservoir (Figure 3).

- Tap the side of the reservoir to encourage any air bubbles to rise to the top of the reservoir (Figure 4).

- Push the plunger slowly to eject the bubbles back into the vial, then slowly pull to draw more insulin into the reservoir. Repeat until the reservoir is filled with the required amount of insulin again (Figure 5).

- Remove the insulin vial from the fill adapter (Figure 6).

- Pull the two release tabs away from the reservoir to release the fill adapter (Figure 7).

- Remove the plunger by unscrewing it counterclockwise (Figure 8).

The fill adapter for 10 mL insulin vial:

- Remove the new reservoir from the packaging.
- Use the plunger to pull the piston to the full position inside the reservoir (Figure 9).

- Use an alcohol wipe to clean the top of the insulin vial and then attach it to the fill adapter (Figure 10).

- Insert the needle into the vial and inject the air, injecting air makes it easier to withdraw insulin from the vial (Figure 11).

- Slowly pull the plunger downwards in the direction of the arrow to fill the reservoir with insulin (Figure 12).

- Tap the side of the reservoir to encourage any air bubbles to rise to the top of the reservoir (Figure 13).

- Push the plunger slowly to eject the bubbles back into the vial, then slowly pull to draw more insulin into the reservoir. Repeat until the reservoir is filled with the required amount of insulin again and there are no any air bubbles (Figure 14).

- Remove the insulin vial from the fill adapter (Figure 15).

- Release the reservoir from the fill adapter (Figure 16).

- Remove the plunger by unscrewing it counterclockwise (Figure 17).

7. Warnings

The Reservoir is a sterile, non pyrogenic disposable product. It has been sterilized using ethylene oxide. Do not reuse or disinfect. Do not use if the package has been opened or damaged.

Please use only Reservoirs that are produced by MicroTech Medical.

Please discard the Fill Adapter in an appropriate sharps container.

Excess air in the Reservoir may cause inaccurate delivery of insulin. Eject as much air as possible. It is suggested that the product should be changed every three days.

8. Notes

Insulin should be at room temperature before filling the Reservoir.

Use immediately after filling. Do not store insulin in the Reservoir when the Reservoir is not in use.

This product should be used under the guidance of doctors or other healthcare professionals. It can be used in a home or hospital environment after proper training. If you are a new user, you should prepare your first reservoir with professional help.

Please read this insert carefully before use and follow the directions. Read the Pump and Infusion Set user guides carefully as well.

9. Storage Condition

This product should be stored in a non corrosive gas, cool, dry well ventilated area. Avoid storing with the chlorinated disinfectant.





10. Symbol list

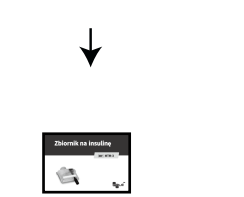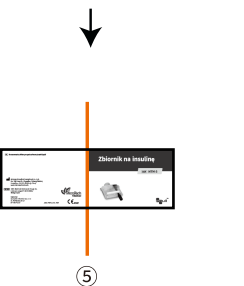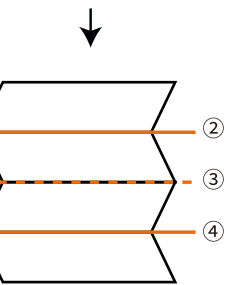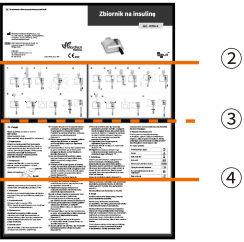Single Use Only	☒
Caution	⚠
Expiration Date	📅
Lot Number	LOT
Sterilized by Ethylene Oxide	STERILIZED
Read Instructions	📖
Do Not Use if Package is Damaged	🚫
Manufacture Date	📅
Manufacturer	🏭

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